

Etac Swift Mobile shower commode

A modern shower commode chair available in several models appreciated for its versatility.



The seat is detachable for easy cleaning.

Various models

Etac Swift Mobile is available in three versions. Swift Mobile, Swift Mobile 160 with increased weight capacity and Swift Mobile 24" which is equipped with 24" rear wheels. All have the same characteristic design and functions.

Back support with maximum comfort

The soft back support is adjustable using velcro straps. The material is carefully chosen so it does not stick to the body. It dries quickly and can easily be removed for cleaning. To prevent the back support from sliding, it is fixed at the top to the bow handle.

Choose seat height

The seat height can easily be set, without using tools. Most common is that the height is set to fit over the toilet. But also consider the user's transfer to and from the chair and the care giver's work situation.



Description

Swift Mobile, part assembled 80209400



Swift Mobile, part assembled w. bucket holder 80209404
Swift Mobile, part assembled w. pan holder 80209405

Standard: back support, seat, arm supports, foot supports and 4 lockable castors

Material

Chassis: powder painted steel
Seat/Arm supports: polypropylene
Back support: polyester/Velcro
Foot support: polypropene
Castors: polyamid/TPE, lockable, ball bearing.

Dimensions

Total width: 58 cm (22¾")
Width between arm supports: 48 cm (19")

Weight

16.5 kg (36.6lbs)



Clean the product with a solvent-free cleaning agent (pH level 5-9), or with a 70% disinfectant solution. Can be decontaminated at max. 85°C / 185°F if so required.

Features




Adjustable back support
The back support can be adjusted using velcro straps.



Ergonomic foot supports
The gently curved foot support ensures comfort and stability.




Easy to adjust the height
Swift Mobile can be adjusted to different fixed seat heights. It is easily done from both sides without using tools.

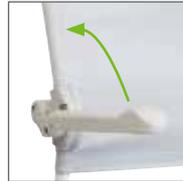

Foldable arm supports
The arm supports are foldable to facilitate side transfer. If needed they can be locked in a folded down position with a locking device (accessory).

Accessories

See our wide range of accessories
"Accessories for all Swift Mobile models"
on page 36

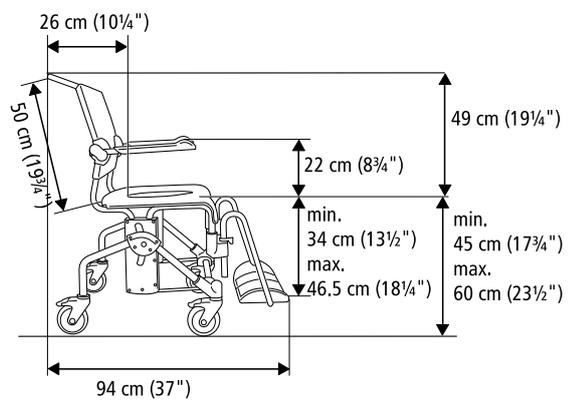

Etac Swift Mobile XL 160

With increased weight capacity and more space between the arm support.



Increased weight capacity

Etac Swift Mobile XL 160 has all the benefits of Swift Mobile; foldable arm supports, adjustable back support, ergonomic foot support and seat height adjustability. In addition Swift Mobile 160 is sturdy enough to handle a user weight of 160 kg (353 lbs).


Easy to customize

All accessories available for Swift Mobile also fit the Swift Mobile XL 160 model.



More space

Swift Mobile 160 has 6 cm (2½") additional space between the arm supports.

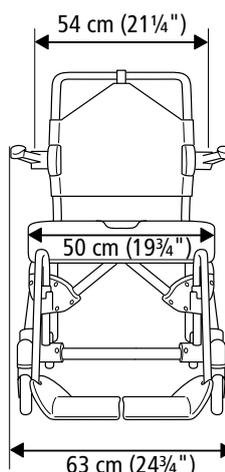

Description

Swift Mobile 160 80209420
 Swift Mobile 160 w. bucket holder 80209421
 Swift Mobile 160 w. pan holder 80209422
 Standard: back support, seat, arm supports, foot supports and 4 lockable castors

Material

Chassis: powder painted steel
 Seat/Arm supports: polypropylene
 Back support: polyester/Velcro
 Foot support: polypropene
 Castors: polyamid/TPE, lockable, ball bearing

Dimensions

For other dimensions see "Etac Swift Mobile shower commode" illustration on page 29.

Weight

17.3 kg (38.1 lbs)



Clean the product with a solvent-free cleaning agent (pH level 5–9), or with a 70% disinfectant solution. Can be decontaminated at max. 85°C / 185°F if so required.



Etac Swift Mobile 24"


A perfect solution for the independent user.

Facilitates side transfers

Etac Swift Mobile 24 is easy to use and propel. The rear wheels are perfectly placed to be easy to reach and give the chair high manoeuvrability.

The seat offers sufficient space in front of the wheels to make a safe side transfer. Swift Mobile 24 also offers seat height adjustment, just as the other Swift Mobile models.

A basic Swift Mobile with four castors can be remodelled to a Swift Mobile 24" by assembling the accessory "Rear Wheel Kit". See page 35.

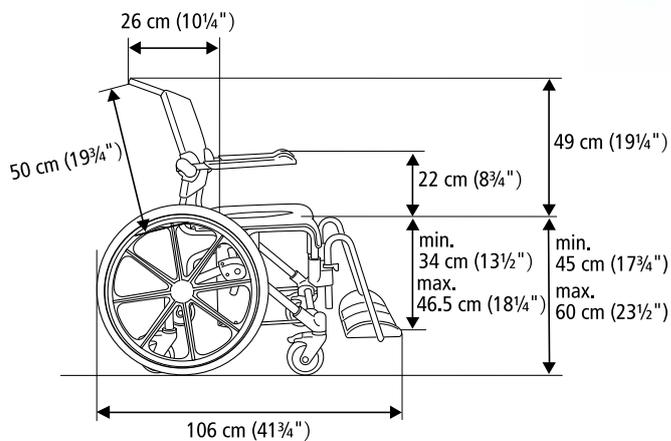

Accessories

See our wide range of accessories
"Accessories for all Swift Mobile models"
on page 36

Description

Swift Mobile 24" assembled	80209402
Swift Mobile 24" assembled, incl. pan holder	80209408
Swift Mobile 24" incl. bucket holder	80209407

Standard: back support, seat, arm supports, foot supports and 4 lockable castors, rear wheels 24", brakes.

Dimensions

Total width: 74 cm (29")
Width between arm supports: 48 cm (19")

Material

Rear wheels: solid tyres, polyamid, ball bearing
Brakes: polyamid, stainless steel
Other material: see page 28

Weight

24.4 kg (53 lbs)



Clean the product with a solvent-free cleaning agent (pH level 5–9), or with a 70% disinfectant solution. Can be decontaminated at max. 85°C / 185°F if so required.



Etac Swift Mobile Tilt

With a unique tilt in space function.

Etac Swift Mobile Tilt is available in two models. Both have manually operated tilt functions but with different weight capacity. They all have the same characteristic design and functions as Etac Swift Mobile, page 28. As an option Swift Mobile Tilt can be equipped with a power unit for the tilt function.

Tilting operated from the side

Etac Swift Mobile Tilt has the unique function which enables tilt manoeuvring from the side. This gives the best conditions for an ergonomically correct work position for the carer. It also enables eye contact and communication with the care recipient during the tilting process.

Assisted restoring to an upright position

The gas pistons work to facilitate bringing the seat into an upright position. When tilting, the user's body weight will act as counterweight and make the movement smooth. This reduces the risk of carers suffering work related injuries. The seat height can be set in various fixed positions and it's done without using tools.



A tilted position increases stability and facilitates in care situations.



Power tilt function for easy and smooth manoeuvre

Etac Swift Mobile Tilt can be equipped with a power unit to control the tilt function which is maneuvered with a practical hand control. This means that the user is also able to control the tilt function. The Power unit occupy very little space, which means that the shower stool can still be positioned over the toilet. The chair is approved to be used in the shower and a separate battery charger is included.

Features



Unique tilt bar

The bar for the tilt mechanism is easy to reach from either side.



Head Support

The adjustability ensures a comfortable and restful support.





Close contact during the tilting process

The unique tilt function enables an ergonomically correct working position for the carer who is able to maintain eye contact with the care recipient.



Power tilt unit

Enables a better working environment. Smooth manoeuvring with a practical hand control.

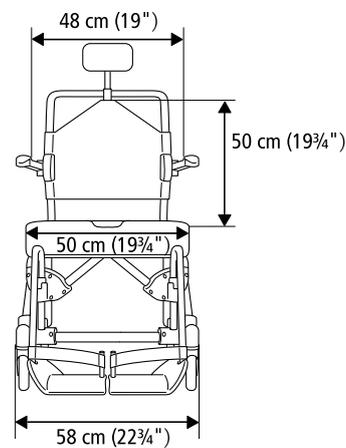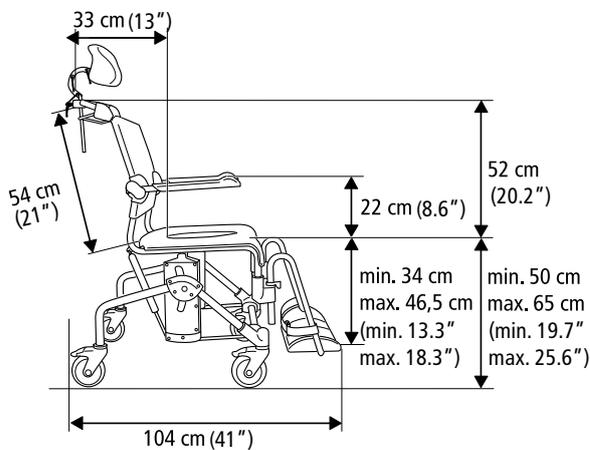

Accessories

See our wide range of accessories
"Accessories for all Swift Mobile models"
on page 36

Description Etac Swift Mobile Tilt

Swift Mobile Tilt	80209410
Swift Mobile Tilt with bucket holder	80209411
Swift Mobile Tilt with pan holder	80209412
Swift Mobile Tilt with power unit	80209490
Swift Mobile Tilt with power unit incl. bucket holder	80209491
Swift Mobile Tilt with power unit incl. pan holder	80209492

Standard: head support, back support, arm supports, seat, foot supports, heels traps, and 4 lockable castors.

Dimensions

Adjustable tilt in space: 0° to -35° can be set to +5° to -30°

Material

Gas pistons: lift capacity 130N
Other material: see "Etac Swift Mobile shower commode" on page 28

Weight

19.7 kg (43.4 lbs)
With power unit: 23.6 kg (52 lbs)



Clean the product with a solvent-free cleaning agent (pH level 5-9), or with a 70% disinfectant solution. Can be decontaminated at max. 85°C / 185°F if so required.

Etac Swift Mobile Tilt XL 160

Swift Mobile Tilt XL 160 has all the benefits of Swift Mobile Tilt and is in addition sturdy enough to handle a weight of 160 kg (353 lbs).

Tilting operated from the side

Swift Mobile Tilt has the unique function which enables tilt manoeuvring from the side. This gives the best conditions for an ergonomically correct work position. It also enables eye contact and communication with the care recipient during the tilting process.

Assisted restoring to an upright position

The gas pistons work to facilitate bringing the seat into an upright position. When tilting, the user's body weight will act as counterweight and make the movement smooth. This reduces the risk of carers suffering work related injuries. The seat height can be set in various fixed positions and it's done without using tools.




Accessories

See our wide range of accessories
"Accessories for all Swift Mobile models"
on page 36



More space

Swift Mobile 160 has 6 cm (2½") additional space between the arm supports.

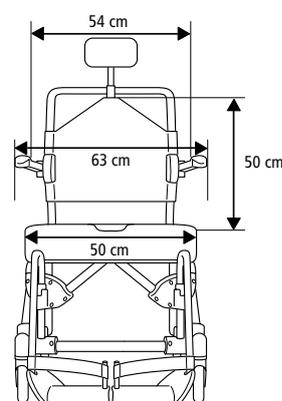

Description

Swift Mobile Tilt 160	80209480
Swift Mobile Tilt 160 with bucket holder	80209481
Swift Mobile Tilt 160 with pan holder	80209482

Standard: head support, back support, arm supports, seat, foot supports, heels traps, and 4 lockable castors.

Dimensions

Total width: 63 cm (24¾")
Total length: 104 cm (41")
Adjustable tilt in space: 0° to -35° can be set to +5° to -30°
Adjustable seat height: 50-65 cm (19¾-25¾")
Width between arm supports Swift Mobile Tilt 160: 54 cm (21¼")

Material

Gas pistons: lift capacity 130N
Other material: see page 28

Weight

20.4 kg (45 lbs)



Clean the product with a solvent-free cleaning agent (pH level 5-9), or with a 70% disinfectant solution. Can be decontaminated at max. 85°C / 185°F if so required.





Accessories for all Swift Mobile models



Head Support

80209447

Complete with attachment. Adjustable in height, depth and sideways. Standard on tilt models.

Material: polyurethane, stainless steel and zinc. Cleaning max. 85° C (185°F) or with alcohol wipe.



Comfort Cover

80209455

A soft back support with integrated arm support pads.

Material: polyethylene.

Cleaning max. 60° C (140 °F) or with alcohol wipe.



Arm supports widening kit

Pair: 80209441

Increases the space between the arm supports by 6 cm (2½"). Standard for Swift Mobile/Tilt 160.



Comfort Kit

80209457

Includes both Comfort Seat and Comfort Cover.

Cleaning max. 60° C (140 °F) or with alcohol wipe.



Hemi Arm Support

Single: 80209035

Fits both left and right side. Material: Cellular polyethylene, aluminium, stainless steel, polystyrene.



Comfort arm pads – pair

80209508

The pads provide a soft support for the arms.

Material: polyethene

Cleaning max. 60° C (140 °F) or with alcohol wipe.



Arm supports lock

Pair: 80209445

Locks the arm supports in down-folded position. Material: Stainless steel.



Side support

80209509

Adjustable in width.

Material: Polyester fabric, Velcro, polyethylene.

Cleaning at max 60°C (140°F)




Soft Back Pad

80209448

Warm, soft and water repellent.

Material: Polyethylene

Cleaning max. 60° C (140°F) or with alcohol wipe

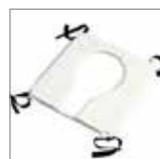

Soft cushion with standard opening

80209065

Material: Polyether/Sealed coated cover




Seat cover

80209434

When a complete seat is required.

Material: Foamed polyurethane.

max. 85° C (185°F) or with alcohol wipe

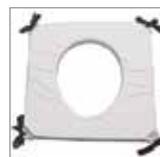

Soft cushion with oval opening

80209067

Material: Polyether/Sealed coated cover



Soft Comfort seat

Opening width 15 cm (6"): 80209507

Opening width 18 cm (7"): 80209506

Material: Polyurethane

Cleaning at max 85°C (185°F) or with alcohol wipe.



Chest harness strap

80209442

Adjustable in length. Attach to back support. Always use together with positioning belt.

Material: Polyester



Cushion, Protect

80209245

Air-inflated ROHO cushion with excellent pressure distribution properties. Adjustable air pressure. Recess width: 21 cm (8¼").

Material: Neoprene

Cleaning max: 60° C (140 °F)



Cross bar

Length: 60.5 cm (24"): 80303034

Length: 66.5 cm (26"): 80209452

For safety and additional support.

Material: Aluminium, polypropene and cellular polyethylene. Notice: 66.5 cm is for Swift Mobile 160 and Swift Mobile Tilt 160.



**Positioning belt**

80209440

Adjustable in length. Attachment with velcro straps.
Material: Polyester**Elevating Leg- and calf support**

80209437

Fits both left and right side.
Material: ABS**Splash Guard**

80209430

Fits into the recess opening. To be placed when user sits in the chair.
Material: Polyurethane**Foot support for leg- and calf support**

80209443

Fits both left and right side. Foot support adjustable height in 6 positions.
Material: Stainless steel, polypropylene.**Attachment for pan/bucket holder**

Set: 80209435

To be mounted underneath the seat.
Material: Polyamide.**Leg support**

1 piece: 80209446

Leg support with foldable and angle adjustable footplate. No heel strap included.

**Pan holder**

80209257

Fits both Pan and regular hospital pans with a thin rim. Attachment required.
Material: Polyamide.**Heel straps**

Pair: 80209432

Adjustable in length.
Material: Polyester**Pan**

80209254

Pan, incl lid with handle

80209255

Material: Polypropylene.

**Anti tippers, pair**

Excluding attachment

for Swift Mobile 24": 80209438

Including attachment

(for Swift Mobile): 80209439

(for Swift Mobile Tilt):80209459

Increases safety if the center of gravity is far back, e.g. leg amputations.

Material: Aluminium

**Bed pan**

82001001

Polyamide, able to withstand boiling.
Fits the Pan holder.**Tilter**

80209449

For easy transfer over thresholds and other obstacles

**Bucket holder**

80209258

Fits bucket with a thick rim. Attachment required.
Material: Polypropene**Bucket with handle and lid**

80302026

Material: Polypropylene


**Rear wheel kit**

80209436

24" rear wheels with solid tires. With built in attachment for anti tip.

Material: Polyamide, stainless steel.

Incl. attachment and brakes. Do not fit Swift Mobile 160 or Swift Mobile Tilt.

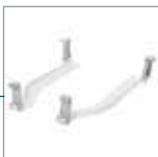**Attachment for low mounting of pan/bucket**

80209259

Fitted to the pan/bucket holder.
Material: Polypropene**Power tilt unit**

80209499

For Swift Mobile Tilt and Swift Mobile Tilt 160 only. Hand control and charger are included.

**Amputee leg support**

80209510

Adjustable height, length and angle. Fits both left and right side.