

Limited Warranty

One (1) Year warranty against defective materials or workmanship on frame structure, 6 monthon components and parts.

This warranty does not cover device failure due to misuse, negligence, improper storage or handling, improper operation and unauthorized modifications. Normal wear and tear on hand grips, brake cables, wheels, and totes are not covered under warranty. These items are subject to normal wear and need periodic replacement and/or adjustment. Please see your dealer for periodic evaluations of your walker.

It is recommended but not required that the walker be returned to the our dealer through whom it was originally purchased. However all our dealers are qualified to assist you in obtaining warranty service. If the walker is to be returned to us prior authorization will be required.

All our products are identified by a seria number at the time of manufacture. Removal or defacing the serial number may void this warranty.



Walkers

Product & Warranty Information

User Instructions

This device was built to exacting standards and carefully inspected to provide support, assistance while walking. This device has a weight capacity of 350 pounds.

Ensure that all black hand knobs are tightened and the hex bolt sits squarely in the frame. For proper adjustment of the handle bars, extend your arm toward the floor, the hand grip should be at wrist height. With your hand on the grip your arm should be flexed 20°-30°. The handle bars should always be adjusted so that the walker height is level.

To fold the 4 wheel walker, lift the seat and you will see a small rope handle at the front of the seat and on top of the tote. Simply lift on the rope handle (without leaning on the walker) and the walker will fold. To unfold simply press the handle grips down. To use the seat on the 4 wheel walker both brakes must be in the locked position to ensure the safety of the user.

Care and Maintenance

- Ensure that all attached hardware is secure at all times.
- Replace any broken, damaged or worn items immediately.
- Clean wheels with warm water and a mild cleanser. Dry with a clean rag.
- Over time the brake cables may stretch with use, they can be adjusted by a qualified dealer. Failure to do this may result in injury to the user.

Precautions

- Do not use on stairs or curbs.
- Watch out for: cords, throw rugs, toys on the floor, spilled water or oil
- Make sure all folding mechanisms are in the locked position when the walker is in use.
- Engage brakes when using seat on 4 wheel walkers.



MERITS HEALTH PRODUCTS CO., LTD.

No.18, Jingke Road, T.P.M.T. Park, Nantun District, Taichung 40852, Taiwan, R.O.C.

TEL: +886-4-23594985 Fax: +886-4-23594992

inquiry@merits.com.tw ; www.meritshealth.com

MERITS HEALTH PRODUCTS, INC.

730 N.E. 19th Place, Cape Coral, FL 33909

TEL: 1-800-963-7487, 1-239-772-0579 Fax: 1-239-574-2661, 1-239-772-3252

www.meritshealth.com